
# Grip-Lok®

REF 3604MCS-TA-1

- EN Catheter Fixation for ARROW® PICC/CVC
- DE Katheter-Fixierung für ARROW® PICC/CVC
- FR Dispositif de fixation de cathéter central à insertion périphérique/cathéter veineux central ARROW® PICC/CVC
- IT Dispositivo di fissaggio per cateteri PICC/CVC ARROW®
- NL Katheterfixatie voor ARROW® PICC/CVC
- SV Kateterfixering för ARROW® PICC/CVC
- ES Fijación de sonda para PICC/CVC ARROW®
- PT Fijação de Cateter para CVC/CCIP ARROW®
- EL Καθίστηξη καθετήρα για ARROW® PICC/CVC
- FI ARROW® PICC-/keskuslaskimokatetrien kiinnityslaitte
- DA Kateterfiksering for ARROW® PICC/CVC
- IS Holleggsfesting fyrir ARROW® PICC/CVC
- NO Kateterfiksering for ARROW® PICC/CVC
- PL Mocowanie do cewników ARROW® PICC/CVC
- CS Fixace katetru pro ARROW® PICC/CVC
- SK Fixácia katétra pre ARROW® PICC/CVC

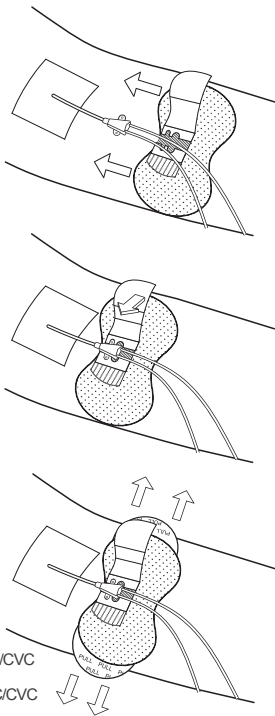44U00038 REV 01

TeleflexMedical Europe  
IDA Business and Technology Park  
Dublin Road, Athlone, Co. Westmeath, Ireland  
Phone + 353 (0)9 06 46 08 00  
www.teleflexmedical.com

## Teleflex®

TIDI Products, LLC  
570 Enterprise Drive  
Neenah, WI 54956, USA  
Phone: + 1.920.751.4300  
tidiproducts.com/securityment

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UK CA 0086  
UK Responsible Person:  
Emergo Consulting (UK) Limited  
c/o Cr 360 - UL International  
Compass House, Vision Park Histon  
Cambridge CB24 9BZ  
United Kingdom

Grip-Lok is a registered trademark of TIDI Products, LLC.  
May be patented: see www.tidiproducts.com/virtual-patent-marking

2023-01-03

### INSTRUCTIONS

EN English

- Intended for use in the general population.
- Select the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
  - Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
  - Open the top flap, remove the interior liner and hold the flap open.
  - Slide the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
  - Secure the top flap over the catheter.
  - Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
- Direction:** For additional skin protection, use skin prep pad prior to application of stabilization device.
- Note:** Use of an alcohol swab may help removal of securement device from the skin.
- ⚠️ If you experience severe redness, itching, swelling, or irritation of the skin, consult your physician as this may be a sign of an allergic reaction.
- ⚠️ Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.
- ⚠️ Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.
- Note:** If being used in a medical facility, dispose according to medical facility and municipal guidelines.
- ⚠️ Do not use if the product's sterile barrier system or its packaging is compromised.
- Report any serious incident related to device use to TIDI Products and the Member State competent authority.

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