AquaGuard®

- (EN) Shower Protection:
Self-Adhesive Moisture Barrier**
- (FR) Protection pour la douche :
barrière autoadhésive contre l'humidité**
- (ES) Protección para la ducha:
barrera autoadhesiva contra la humedad**

<http://www.tidiproduts.com/ifu>

TIDI
Support Caregivers.
Protect Patients.

EN INSTRUCTIONS FOR USE

Description

AquaGuard is an impermeable moisture barrier that allows a clinician or patient to cover wounds, dressings and intravenous sites and keep them dry when showering. It consists of an occlusive polyethylene barrier with a gentle adhesive border, and is available in a variety of sizes.

Indications for Use

AquaGuard is designed to cover and keep wounds, dressings and intravenous sites dry during showering. Common applications include PICC lines, ports, subclavian and internal jugular sites, small surgical sites, and other sites on the chest or back. This product may be applied by healthcare providers or patients.

Contraindications

AquaGuard is intended for showering only and is not recommended for swimming or submersion underwater. *AquaGuard* is not a replacement for primary dressings.

Precautions

For best results, a caregiver should apply the *AquaGuard* product to the patient. Apply *AquaGuard* to clean dry skin. Do not use lotion or cream before applying. Do not lift and/or attempt to reposition *AquaGuard* after placement. Do not use if punctured or showing signs of wear. *AquaGuard* is a single use product. If reused, may not provide an effective barrier while showering. If sensitivity or irritation develops, discontinue use.

Ensure proper size *AquaGuard* is selected prior to application. For a detailed list of available sizes and coverage areas, visit <http://www.tidiproducs.com/ifu>

Directions For Application

1. Prior to application, keep the skin dry and clear from oils and lotion. An alcohol wipe may be used to prepare skin for application.
2. Prepare *AquaGuard* for application by forming "tabs" in the backing strips at each corner. See **Figure 1**.
3. Using the tab, remove the top backing strip and apply the underlying adhesive to the skin. See **Figure 2**.
 - **For subclavian sites:** Place the adhesive on skin above the central line or port dressing.
 - **For internal jugular sites:** Place the adhesive just above the dressing following below the contour of the jaw line and if possible toward the chin.
4. Individually remove the side backing strips, pleat each side, and apply. See **Figure 3**.
 - **For subclavian sites:** Remove the side strip and create a "pleat" in the middle while placing adhesive on skin. Pinch the pleat then fold side-to-side while applying pressure to ensure good adhesion. Repeat for other side.
 - **For internal jugular sites:** Have patient tilt neck away from site. Remove side strip and place along side of the dressing site. If pleating is necessary, pinch the pleat then fold side-to-side while applying pressure to ensure good adhesion. Repeat for other side.
5. Remove the bottom strip and place just below dressing site, pleating if necessary. See **Figure 4**.
6. Massage adhesive border against skin to ensure seal. See **Figure 5 and 6**.

Directions For Removal

1. After shower, gently pat surrounding area dry before removal.
2. Gently pull down in the direction of hair growth and discard.

Note: Some dampness may occur due to perspiration.

Figure 1

Figure 2

Figure 3

Figure 4

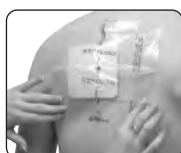
Figure 5

Figure 6

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© 2021 TIDI Products, LLC
AQUAGUARD® is a trademark of TIDI Products, LLC, registered in the USA. Used under license.
TIDI Products, LLC